



The trauma clock specially designed for medical trauma rooms.

CHRONOFLEX

Clock


Shows time in your own chosen time zone(s). NTP-synchronized, with clear dot ring.

Remote

Using the supplied simple and intuitive remote control the system is easily controlled. (start, stop, pause).

Colors

The colors ensure an easy overview of the time and status.

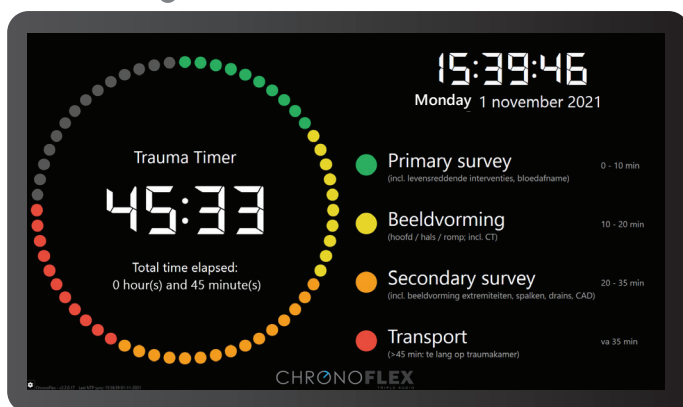

Speakers

The speaker that is standard included makes audibly clear what the status is.

View

The high-quality and clearly visible 24" screen provides a clear visual representation of the ChronoFlex system.

Timers

The flexible nature of ChronoFlex provides a variety of functionality:

- Trauma timer
- Resuscitation timer
- NTP-synchronized time display

Collaboration

The audiovisual support is created through collaboration with trauma physicians and users from the leading medical centers of the Netherlands.



Specifications


Trauma Timer

- The ring fills in one hour.
- The ring has colored dots that indicate different periods.
 - 0-10
 - 11-20
 - 21-35
 - 36-59
- The elapsed time of both the trauma timer and the system are displayed as Hours : Minutes : Seconds in the center of the clock.
- You hear speech when the timer is started / stopped / paused.
- You hear speech when switching periods. For example "Ten minutes have passed."
- The trauma timer can be controlled remotely: Start / Pause / Stop (double tap is reset).



Resuscitation Timer

- The ring fills in two minutes.
- The ring has colored dots that indicate different periods.
 - 0-10
 - 11-20
 - 21-35
 - 36-59
- The elapsed time of both the resuscitation timer and the system are displayed as Hours : Minutes : Seconds in the center of the clock.
- You hear speech when the timer is started / stopped / paused.
- You hear speech when switching periods. For example "Ten seconds left."
- The resuscitation timer can be controlled remotely: Start / Pause / Stop (double tap is reset).

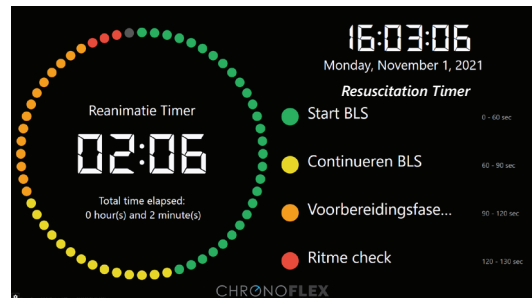

NTP-synchronized time display

- The ring fills in one minute.
- The dots have a blue color.
- The date is displayed in writing.
- The time is displayed in the selected time zone which is to be set under the time display option.
- The clock is NTP-synchronized.



Curious?

Contact us and request a demo!